

INSTRUCTIONS FOR USE

Product Name: Particle Filtering Half Mask

Model Number: G95200

Classification: KN95

Trademark : **GCPC** GCPC

Type: Foldable, Ear loops

Size: 16cm*10cm

Colour and Structure: White, Four-layer structure (outer layer PP nonwoven fabric + middle layer high filter efficiency meltblown nonwoven fabric + meltblown nonwoven fabric + skin friendly nonwoven fabric),
Nose clip and Ear loops.

Executive Standard: GB 2626-2006



HUIZHOU GREEN COMMUNICATION EQUIPMENT MANUFACTURING CO., LTD.
No.3, Hengyu road 1, Sandong digital industrial park, Huizhou city, Guangdong province,
China

Intended use:

It is intended to cover the nose and mouth of the wearer to help reduce wearer exposure to airborne particulates and also protects the wearer from splash and spray of body fluids.

Using Scope And Limitations:

1. This mask is suitable for protection of non-oil particulate matter. e.g. dust, pollen, industrial dust, air pollution fine particles (PM2.5).
2. This mask is not suitable for protecting harmful gas and steam; it is not suitable for underwater operation and anoxic environment; it is not suitable for the environment where oil mist exists.
3. This mask is can not be used in environments where the concentration of hazardous substances reaches immediate life- and health-threatening (IDLH) concentrations, in environments that are not hazardous to the unknown, in occupational health exposure concentration limits (OEL) where the concentration of hazardous substances exceeds 10 times or above the limits prescribed by the local government.
4. This mask can not produce oxygen and must not be used in an environment where oxygen content is less than 19.5%.
5. This mask is suitable for adults and not for children.

Check Before Use:

1. Check the overall appearance of the mask before use should be good, all parts of the mask should not be damaged or seriously contaminated, ear loops should be free of breakage, good elasticity, nose clip should not be damaged or broken.
2. Do not use the mask when it is contaminated or damaged. Replace a new mask.

Wearing Instructions:

1. Pull both ear loops and open folded masks;
2. Place the chin inside the mask;
3. Pull the ear loops to the ear;
4. Place the fingertips of both hands on the top of the metal nose clip, and the fingertips slide along the sides of the metal strip to shape the nose area into the shape of the nose;
5. Cover the mask with both hands and breathe quickly. If there is air leakage around the mask, please continue to adjust the position of the mask, nose clip or ear loops tightness until a proper seal is achieved.



Checking Your Seal:

1. Place both hands over the mask, take a quick breath in to check whether the mask seals tightly to the face.
2. Place both hands completely over the mask . If you feel leakage, there is not a proper seal.
3. If air leaks around the nose, readjust the nose clip according to the wear method 4; If air leaks at the mask edges, re-adjust the ear loops along the sides of your head until a proper seal is achieved.
4. If you cannot achieve a proper seal due to air leakage, ask for help or try a different size or model.

Service life and judgment methods:

1. When the mask is contaminated or the breathing resistance is obviously increased during use, please discard and replace a new mask immediately.
2. When the mask is damaged or deformed by external force, the mask can not close well with the face, please discard and replace a new mask immediately.
3. In case of visible dirt, wetness or residual moisture, or foreign matter such as blood stains or droplets, please discard and replace a new mask immediately.

Storage Conditions and Shelf Life:







1. The mask should be stored in a clean, ventilated, dry, oil free, free of direct sunlight and non-corrosive gas and away from open fire.
2. If the mask is not used frequently, it is recommended to put the mask into a sealed bag for storage, and the mask deformation should be avoided when stored.
3. Storage temperature range : 0°C to 35°C; relative humidity :<85%.
4. Shelf life: from the production date, the shelf life is 2 years if the above conditions 3 storage temperature and humidity requirements are met.

Warnings

WASH YOUR HANDS THOROUGHLY BEFORE PUTTING ON AND TAKING OFF THE MASK.

1. Before use the masks first verify that it is suitable for the intended use. The wearer must be adequately trained prior to use and ensuring the masks are proper fit.
2. The mask is disposable and cannot be reused.
3. Do not allow facial hair, hair, jewelry, glasses, clothing, or anything else to prevent proper placement or come between your face and the mask.
4. Do not touch the front of the mask during or after use. It may be contaminated.
5. The external force acting on the metal nose clip may cause injury to the wearer. Please pay special attention in running, cycling or other situations.
6. Patients with heart disease or other disease should discard it after wearing uncomfortably.
7. It is strictly prohibited to use after the package is damaged.
8. Wrong use may cause illness or even death. Materials in direct contact with the skin may cause allergic reactions in some sensitive individuals.

Explanation about sign, symbols:

	Keep dry		Please put the cartons up
	Stack no more than 5 layers		Use no hooks
	Manufacturer		Trademark